DEI® CERAMIC PRIME

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DEI® CERAMIC PRIME

PRIMING AGENT
FOR PRIMING CERAMICS

Product for dental use
Keep away from children
Date of the specifications: 07/2019

Available packages
One 5 ml bottle **REF BOND 24**

Characteristics

DEI® Ceramic Prime is a simple to use and effective one component agent for priming ceramics, porcelain and as well non-precious metals.

Composition

Solution of methacrylsilane in ethanol.

Technical data

Adhesive strength:
- Ceramic surfaces: 17 MPa
- Surfaces of non-precious metals: 15 MPa

Way of use

Isolation

Use of rubber dam is strongly recommended during intraoral repair of the prosthetic work.

Pretreatment of the surfaces of the prosthetic works

After cleaning the areas to be primed thoroughly, roughen the surface with a diamond bur (40 µm) or - if indicated - with a suitable intraoral sand blasting. Then rinse with plenty of water and dry with oil and waterfree air.

If the application can be done extraorally, ceramics or porcelain can be etched with hydrofluoric acid. This results in an enhanced bond strength.

After etching rinse with plenty of water. Do never use it intraorally.

Application of DEI® Ceramic Prime

DEI® Ceramic Prime is put in a mixing well and applied immediately in a thin layer with a brush onto the prepared surface. Dry gently with oil-free air until the surface is no longer moist and appears glossy. Thus, the prepared areas are ready for bonding with acrylic resins. This has to be done within 30 minutes.

Suitable acrylic resins are bonding resins, composites, flowable composites or composite cements.

Important note

DEI® Ceramic Prime can interfere with bonding to dentin. Therefore, when applying silane, use a fine brush and limit its application to the restorative material. Do not allow DEI® Ceramic Prime to run onto the adjacent tooth structure.

Warnings

Hydrofluoric acid is strong corrosive and toxic! Avoid strictly any body contact (e.g. hands)!

Conservation

Do not store above 25°C (78°F). Do not use after expiry date.

Via Torino 765 - 21020 - Merello (VA) - Italy
Phone +39 0331 969270 - Fax +39 0331 969271
www.deiitalia.it

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