DEI® Rainbow Fit Crown

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DEI® Rainbow Fit Crown

**VINYLPOLYSILOXANE
FOR CONTROL
OF FRICTION POINTS**

Available package

Two 50 ml cartridges
(COD. FIT 1)

Product for dental use.
Keep out from children.
Date of specifications: 06/2006

Characteristics

DEI® Rainbow Fit Crown is a vinylpolysiloxane specifically designed for control of friction points in prosthetic crowns and inlays.

It can also be used for control of pressure points of partial and total mobile prosthesis.

Advantages

It has a high shore A (80) which allows the milling of contact points without block of processing by the product. Times of processing optimal even for a full arch. The ideal consistency allows to insert the prostheses without any resistance. High opacity to highlight areas of friction.

Way of use

The product is auto-mixing. After positioning of the cannula on the cartridge dispense directly in situ. Place the prosthesis in the oral cavity within 50 seconds. Wait for the completion of curing, about two minutes. Leave the cannula inserted used until next use.

Cautions

The times of processing and curing are highly influenced by temperature of storage and use. High temperatures shorten times, low temperatures lengthen them.

Storage and shelf life

Store at room temperature not exceeding 28°C.

DEI® italia

Via Torino, 765 - 21020 Mercallo (VA)
Phone +39 0331 969270
Fax +39 0331 969271
www.deiitalia.it