DEI® Rainbow Fast

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VINYLPOLYSILOXANE FOR PRECISION IMPRESSIONS

Product for dental use only. Keep away from children. Date of specifications: 11/2010

Available packages

DEI® Rainbow Fast Light: four 50 ml cartridges (**COD. RA2 2**)

DEI® Rainbow Fast Medium: four 50 ml cartridges (**COD. RA2 3**)

Characteristics

DEI® Rainbow Fast Putty, Light and Medium, are highly hydro-compatible vinylpolysiloxanes, with ideal viscosity for all techniques of impression.

DEI® Rainbow Fast Putty, Light and Medium comply with ISO 4823 regulations.

Composition

Vinylpolysiloxane materials.

Way of use

Insert the mixing cannula on the cartridge and extrude directly into the impression a suitable amount of fluid, avoiding incorporation of air bubbles. After use, leave the cannula inserted on the cartridge up to next use for avoiding eventual contaminations of the product, with consequent obstruction of outlet holes.

Indicative working times (DEI® Rainbow Fast Putty)

Mixing: 30/45 seconds (up to achieve homogeneous coloration of the mixture)

Working: 1.30 minutes

Permanence within oral cavity: 2.30 minutes

Hardening: 4 minutes

Indicative working times (DEI® Rainbow Fast Medium and Light)

Mixing: Automix

Total working: 1 minute

Permanence within oral cavity: 2.00 minutes

Hardening: 3 minutes

Warning

The impression can be cast after 2 hours, but it is advisable to wait 12 hours and not go beyond 7 days.

The impression can be electroplated in copper or silver. Working and hardening times refer to use at room temperature. Higher temperatures shorten times, lower ones lengthen them.

Notes

The use of the products require dispenser gun and cannulae, not included.

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