DEI® Rainbow Hydro Big Putty

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DEI® Rainbow Hydro Big Putty

VINILPOLYSILOXANE
FOR UPMOST HYDROCOMPATIBLE
PRECISION IMPRESSIONS

Product for dental use
Keep away from children
Date of the specifications: 11/2019

Available packages

Two 380 ml cartridges + 10 cannulas **REF RAINH 9**

Characteristics

DEI® Rainbow Hydro Big Putty is a high viscosity material based on vinylpolysiloxane technique packed in cartridges for mixing devices.

Together with DEI® Rainbow Hydro Light or DEI® Rainbow Hydro Medium, it is recommended in technique of double impression or single impression double component and also in Twin technique.

It conforms to ISO 4823 type 0 regulation.
Shore A about 69.

Advantages

DEI® Rainbow Hydro Big Putty shows high dimensional stability and best elastic memory, is odourless, tasteless, has an high tear resistance.

Instructions

Extrude the product on the impression tray. Insert the impression tray into the oral cavity within 1 minute from the extrusion, and let it in place at least 2 minutes.
The indicated times refer to room temperature of 23°C.
Higher temperature shorten the times, lower ones lengthen them.

Warnings

The impression can be cast 30 minutes after the taking away from oral cavity.
It can be electroplated in copper or silver.
Disinfect with suitable solutions. DEI® italia Sicursept is recommended.
Use specific cannulas for every type of cartridge.

Preparation of the cartridge

Take away the safety cap, insert the cannula.
Block turning the supplied nut half a turn, insert the cartridge into the mixing device.
Don't use again the safety cap.
After use leave the cannula on the cartridge.
Only for new cartridges: for better safety, discard the first 2-3 cm of dispensed compound.

Via Torino 765 - 21020 Mercurio (VA) - Italy
Phone +39 0331 969270 - Fax +39 0331 969271
www.deiitalia.it

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