DEI[®] Rainbow Hydro Light

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Date of the specifications: 06/2007

Characteristics

DEI® Rainbow Hydro Light is a vinylpolysiloxane with very high hydro-compatibility (contact angle with water at 2 sec., 45°) which ensures the perfect reproduction of sub-gingival areas even in the presence of moisture.

It is indicated for taking precision

impressions in the technique of double impression.

DEI® Rainbow Hydro Light is a material with low viscosity, thixotropic (it remains stable without seeping but increases its fluidity when compressed).

It complies with ISO 4823 Type 3 regulations.

Shore A about 45.

Way of use

DEI® Rainbow Hydro Light is packaged in a security cartridge.

Place the cartridge on the dispenser and lock it.

Insert the cannula and block it with a fourth turn.

Extrude the product on an impression tray or directly using an intraoral tip.

Insert the impression tray in the oral cavity within 2 minutes and 15 seconds from the dispensation and keep it there for 3 minutes.

The time reported is indicative: temperatures below 23°C

lengthen it, higher temperatures shorten.

After use leave the cannula on the cartridge till next use.

Advices

Dispense with constant pressure without variations maintaining the cannula in contact with the basis of the impression to avoid incorporation of air; it is suggested to discard the first 2-3 mm of extruded product that could be not fully blended. The impression can be cast after 30 minutes after the removal from the oral cavity. It can be electroplated with copper or silver. It can be sanitized in suitable solutions.

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