## DEI® LAB ISOFILM ISOLANTE SILICONICO

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# DEI® LAB ISOFILM SILICONIC INSULATOR

Product for dental use  
Keep away from children  
Date of specifications: 04/2019

**Available package**  
Two 10 ml bottles (Base + Catalyst) **REF ISO 1**

## Characteristics

DEI® Lab Isofilm is a new two component insulator based on silicone by means of addition. Unlike common insulators, it shows great resistance to heat and liquids. It forms a tenacious film that preserves the integrity of the master model. It owns easy application and always allows perfect removal of contact material. Isofilm is especially recommended for the techniques Experience/Metal Free, Future Wax, self-curing calceinable resins and cold resins (Preceasy).

## Composition

Material based on poly-addition.

## Way of use

Mix in a ratio of 1:1 with little spatula for 15 seconds. Put on the surface to isolate with a get-away little brush and wait the curing (5-7 minutes). If it is necessary to use the Opaque, clean very gently with ethylic alcohol.

## Use for direct inlay technique

1. Prepare the cavity paying attention to avoid making of undercuts.
2. Isolate with dam.
3. Mix two drops of Isofilm base with two drops of Isofilm catalyst for 15 seconds.
4. Put the blend on with small brush and spread paying attention to avoid surpluses.

5. Wait 4-5 seconds.
6. Put the composite on, and light cure.  
At the end of the restoration, extract the inlay.
7. Remove the film of Isofilm using the small brush with goat hair.
8. Etch, apply Extra Bond dual, put Poker Cem Automix on, according with the instructions of use for both products, and insert the inlay.
9. Light cure.

## Use for relining temporary teeth

Using only the liquid of a bottle (red or black lid) it is possible the isolation of prosthetic stumps during the relining of temporary teeth by means of self-polymerizing resin, obtaining a surprising result, for both precision and respect of resin polymerization (the vaseline produces deterioration of the resin polymerization, the silicon is a fully inert material).

## Use in endodontic works

Run inside the endodontic canal before the setting of the resin (technique for making glass fiber stump post) it allows the extrusion of the resin post from the same canal (use only the liquid of a bottle, red or black tip).

## Cautions

For isolation of stumps, it is recommended to begin the application of Isofilm from the occlusal side towards the closing area. Working times are closely related to the temperature of room stocking.  
Keep the bottle well sealed after use.

## Storage and shelf life

Store at temperatures not higher than 25°C.

Via Torino 765 - 21020 Mercallo (VA) Italy  
Phone +39 0331 969270 - Fax +39 0331 969271  
www.deiitalia.it

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