DEI® Rainbow Hydro Mono

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Polydimethyl polymethyl hydrogen siloxane, polydimethyl polymethyl vinylsiloxane, silica, paraffine.

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DEI® Rainbow Hydro Mono

HIGH VISCOSITY MATERIAL
FOR PRECISION
IMPRESSIONS

Product for dental use only
Keep away from children
Date of the specifications: 01/2013

Available package
four 50 ml cartridges (COD. RAIN 16)

Characteristics

DEI® Rainbow Hydro Mono is a single phase material based on vinyl-polyisiloxane. The viscosity of DEI® Rainbow Hydro Mono has been taken at the optimum for assuring the

best making of impressions using single phase technique.

Indeed, its consistency is low enough for detection of excellent details, however sufficient for avoiding overflowing from the impression tray.

It is indicated in preparations of impressions for bridges, crowns, inlays, prosthetic restorations and impressions using copper rings and individual spoons.

Composition

Polydimethyl polymethylhydrosiloxane, polydimethyl-polymethyl-vinylsiloxane, silica, paraffin.

Technical data

Mixing time: auto mix
Total working time: 2'00"
Hardening in the oral cavity: 2'30"
Total setting time: 4'30"
Size change after 24 h: 0,05-0,3%
Elastic recovery: ≥ 99,2%
Strain in compression: 2.6%
Shore A hardness (1 h): about 60
Shore A hardness (24 h): about 63

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Eliminar el primer milímetro del

Way of use

Insert the cartridge on the dispenser gun, pull out the protective cap, insert the mixer cannula and clamp it making one fourth turn of the blocking nut.

Discard the first millimetre of dispensed product; leave the mixer cannula inserted on the cartridge till next use.

Extrude the product on the impression tray and within 2 minutes introduce the spoon in the oral cavity. The product hardens after permanence of about 3'15" in oral cavity. The above mentioned times refer to use at 23°C. Lower temperatures require longer times.

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