




dream frame


monolithic
composite
method

NUOVA MUFFOLA DF/MCM

La nuova muffola DEI® italia è veramente un passo avanti per la realizzazione di strutture in fibra di carbonio Dream Frame / M.C.M.®

È più alta e ha più spazio all'interno dove ora trova spazio sufficiente anche un modello di grandi dimensioni.

È autocentrante e molto precisa.

La cornice superiore e il coperchio sono forniti anche in policarbonato trasparente, materiale molto resistente e adatto alla fotopolimerizzazione.



(COD. DF M)



ISTRUZIONI PER LA MANUTENZIONE DELLE MUFFOLE

- 1) Lavare la muffola con acqua tiepida e sapone per piatti.
- 2) Il plexiglas e il policarbonato non sopportano temperature superiori ai 90°C, in caso la vostra unità fotopolimerizzatrice superi i 90°C per 8 minuti, effettuare due cicli da 4 minuti.
- 3) Nel caso non si possieda un apparecchio a luce con ventola di raffreddamento, occorre non fare fotopolimerizzazioni ripetute. Negli intervalli aprire completamente la porta soffiando all'interno della camera con una pistola ad aria compressa.

MAINTENANCE INSTRUCTIONS FOR FLASKS

- 1) Wash the flask with warm water and dish soap.
- 2) Plexiglas and polycarbonate do not tolerate temperatures above 90°C. If your light curing unit exceeds 90°C for 8 minutes, do two cycles of 4 minutes each.
- 3) If you do not own a light unit with cooling fan, it should not do repeated light cures. In the breaks, open the door completely blowing inside the chamber with a compressed air gun.

