# DEI® EXPERIENCE POLISH PASTE

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Fax +39 0331 969271

[www.deiitalia.it](http://www.deiitalia.it)

**DEI® EXPERIENCE**  
**POLISH PASTE**  
PASTE FOR POLISHING COMPOSITES

Product for dental use  
Keep away from children  
Date of specifications: 11/2017

**Available package**

DEI® experience Polish Paste contains: **REF** PP 1  
one 35 g tube

**Characteristics**

DEI® experience Polish Paste is specifically prepared for polishing composite materials. It contains a filler of about 0.3 microns size that polishes without damage of surfaces and alterations of ratios between filler and binder.

**Composition**

Aluminium oxide, water, glycerine, sweeteners.

**Way of use**

Adoption of DEI® italia's polishing protocol is recommended:

1. Photo-polymerize the last layer of composite after having put DEI® experience Polish Paste on for guarantee of perfect photo-polymerization and elimination of surface inhibition layer.
2. Finish the surface with fine and extra-fine grain diamond burs.
3. Put a thin layer of DEI® Seal Coat Fast on, wait 30 seconds and photo-polymerize for 40 seconds.
4. Apply and spread DEI® experience Polish Paste using a small brush made of goat hair, at a very low number of turns.
5. Run a simple cotton wad for the last polishing.
6. ONLY FOR LABS: repeat steps 4 and 5.

**Cautions**

Don't ingest.  
Keep the tube well sealed after use.

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