DEI® EXPERIENCE POLISH PASTE

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DEI® EXPERIENCE POLISH PASTE PASTE FOR POLISHING COMPOSITES

Product for dental use
Keep away from children
Date of specifications: 11/2019

Available package

DEI® experience Polish Paste contains: **[REF] PP 1**
one 35 g tube

Characteristics

DEI® experience Polish Paste is specifically prepared for polishing composite materials. It contains a filler of about 0.3 microns size that polishes without damage of surfaces and alterations of ratios between filler and binder.

Composition

Aluminium oxide, water, glycerine, sweeteners.

Way of use

Adoption of DEI® italia's polishing protocol is recommended:

1. Photo-polymerize the last layer of composite after having put DEI® experience Polish Paste on for guarantee of perfect photo-polymerization and elimination of surface inhibition layer.
2. Finish the surface with fine and extra-fine grain diamond burs.
3. Put a thin layer of DEI® Seal Coat Fast on, wait 30 seconds and photo-polymerize for 40 seconds.
4. Apply and spread DEI® experience Polish Paste using a small brush made of goat hair, at a very low number of turns.
5. Run a simple cotton wad for the last polishing.
6. ONLY FOR LABS: repeat steps 4 and 5.

Cautions

Don't ingest.
Keep the tube well sealed after use.

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