DEI® Rainbow Putty RT

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DEI® RAINBOW PUTTY RT HIGH VISCOSITY VINYL POLYSILOXANE FOR TECHNIQUE TWIN

Product for dental use only
Keep away from children
Date of specifications: 02/2019

Available packages

One 400 g jar of base paste, one 400 g jar of catalyst paste **REF TWIN 1**

Characteristics

Material for first impressions based on vinyl polysiloxane. It meets the requirements of ISO 4823 type O regulations.

Advantages

- High dimensional stability
- Hydro-compatible
- Best consistency, does not stick and easy to mix
- Odourless and tasteless

Technical data

Shore A hardness after 24 hours: about 62
Linear retraction: < 0.2%
Elastic recovery: 99.4%

Way and times for use

Take with the special spoons equal parts of basis paste and catalyst paste, mix them 40 seconds, without putting latex gloves on. Insert the material into the oral cavity within 1 minute and 30 seconds from the beginning of mixing; let there 3 minutes up to full cure.

Warnings

Don't contaminate the basis paste with the catalyst one. Seal well the lids soon after every taking out of products. Don't invert the lids. Don't invert the use of the spoons (white for the catalyst). The impression can be cast 30 minutes after cure. The impression can be treated with the special disinfectant solutions. Don't use coupled with silicones for condensation or other categories in general. The times indicated refer to use of the product at 23°C and 50% relative humidity. Lower temperatures lengthen these times, higher ones shorten them. Silicones for addition don't have contraindications, however individual sensitivity to components cannot be excluded. In such case, don't use.

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