DEI® SIMPLY CORE HARD

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DEI® SIMPLY CORE HARD

DUAL CURE COMPOSITE
FOR CORE BUILD UP
AND CEMENTATION OF POSTS

Product for dental use
Keep away from children
Date of the specifications: 06/2012

Available package

One 25 ml cartridge, A2 color (**COD. CORE 10**),
10 mixer cannulas and 10 intraoral tips.
Refill 50 mixer cannulas (**COD. CORE 6**)
Refill 50 intraoral tips (**COD. CORE 8**)

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Mixture of poly and difunctional methacrylates, barium glass filler, fumed silica, initiator, catalyst, stabilizer.

Technical data

Compressive strength: 350 MPa
Flexural strength: 140 Mpa
Radiopacity: > 200%
Linear polymerisation shrinkage: < 0.8%
Barcol hardness: 75
Total filler: 66% on weight
Polymerization without light: about 3-4 minutes
Working time: about 1 minute

Way of use

Preparation of stumps

Application of a matrix is recommended:

- 1) Apply DEI® Extra Bond mixed with DEI® Extra Catalyst (1:1) on the left slightly wet dentinal surface and on the post.
- 2) Pull out the syringe's protective plug and insert the cannula with a rotation of 90°. The cannula must be left on the cartridge till next use.
- 3) Put DEI® Simply Core Hard on and adapt it within one minute from the application; light cure for 40 seconds. Without light cure, a complete polymerization takes place in 3-4 minutes. If a preformed crown is used, it is advisable the direct insertion of the product within the crown, its application and keeping in position till full hardening.

Cementation of posts

- 1) Prepare the canal, etch and dry.
- 2) Rinse, clean the post with ethylic alcohol; if the post is made of glass fibre, treat it with DEI® Poker Ceramic Prime for increasing adhesion; let dry.
- 3) Put DEI® Extra Bond on inside the canal and on the post; light cure both for 20 seconds.
- 4) Apply DEI® Simply Core Hard inside the canal and insert the post. Keep under pressure till complete hardening, or light cure for speeding up the reaction.

Cautions and contraindications

For the first use we suggest to discard the initial extrusion (3 mm).

In case of preparation of provisionals using composite materials, it is necessary to isolate the preparation by means of suitable materials like, for example, vaseline.

Don't apply DEI® Simply Core Hard either at the same time, or in areas where eugenol was used.

The above mentioned times are indicative and may also considerably vary depending on the temperatures of storage and area of application of the product.

Avoid contacts between product, skin and eyes. In case of contact with eyes, wash with plenty of water and call for an oculist.

Use of latex dam is advisable.

Today side effects are unknown. In case of individual hypersensitivity, don't use.

Storage and shelf life

Store in freezer the product not yet used or already opened. Opened and daily used, it must be kept at temperature under 20°C.

Before using, bring it to room temperature.

The cartridge must be used within 3 months after the opening.

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