**PRIMER UNIVERSALE
PER TRATTAMENTO OPACIZZANTE
SU DREAM FRAME E METALLO**

CE
0425

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CE
0425

Product for dental use
Keep away from children
Date of specifications: 06/2019

Available packages

- DEI® UniAdhesive 1 **REF UA 1**
One 5 ml bottle
- DEI® UniAdhesive 2 **REF UA 2**
One 3 g syringe

Characteristics

DEI® UniAdhesive 1 is an adhesion promoter for Dream Frame carbon fiber or metal. Read the application protocol carefully before use.

Way of use

Use this application protocol for the following techniques:

- DEI® experience composite on Dream Frame carbon fiber.
 - DEI® experience composite on metal.
1. Sandblast the carbon fiber or the metal

- with 120/160 micron aluminum bixido.
- 2. Clean the carbon fiber with 94°C ethyl alcohol. Clean the metal with steamer.
- 3. Once dry, apply DEI® UniAdhesive 1.
- 4. 2 minutes later, dry with a light blow of uncontaminated air.
- 5. Apply DEI® UniAdhesive 2 and light cure for 2 minutes.
- 6. Apply DEI® Dream Opaque Wash Opaque and light cure for 4 minutes.
- 7. Apply DEI® Dream Opaque Universal Opaque and light cure for 4 minutes.
- 8. Apply DEI® experience composite.

Storage

DEI® UniAdhesive 1 should be stored at 10-25°C. Close the bottle properly immediately after use and protect against the effects of direct light.

Shelf life

The maximum shelf life is printed on the label of each bottle. Do not use after the expiry date.

Guarantee

Our technical instructions, regardless of whether they are provided verbally, in writing or during practical demonstrations, are based on our own experience and should only be considered guidelines. As our products are subject to continued development, we reserve the right to modify them. Complies with EN ISO 10477.

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Note

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects - even if it is doubtful that the side-effect has been caused by our product - please kindly contact us.

Contra-indications / Interactions

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

Via Torino, 765 - 21020 Mercallo (VA) - Italy
Phone +39 0331 969270
Fax +39 0331 969271
www.deitalia.it

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