DEI® CLEVER EASYFLOW

COMPOSITO FLUIDO MICRO IBRIDO

CE
0546

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DEI® Clever Easyflow risponde alle norme ISO 4049.

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Bisphenol A Diglycidyl Methacrylate, Diurethane Dimethacrylate, Triethylene Glycol Dimethacrylate and Ethoxylated Bisphenol A Dimethacrylate, inorganic fillers: Barium Aluminio Silicate Glass, Fumed Silica, Titanium Dioxide and additional substances (photoinitiators, inhibitors, catalysts, stabilizers and pigments) become as a result of free radical polymerization, activated by the ultraviolet light (400nm-500nm).

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DEI® CLEVER EASYFLOW

MICRO-HYBRID FLUID COMPOSITE

Product for dental use only.
Keep away from children.
Date of specifications: 08/2017.

Available packages

Two 3.4 g (2 ml) syringes and 6 tips.

Available in the colours:

A2 (REF FLOW 1), A3 (REF FLOW 2),
A3.5 (REF FLOW 9), A4 (REF FLOW 3),
B2 (REF FLOW 5), C2 (REF FLOW 6),
INCISAL (REF FLOW 8), BLUE (REF FLOW 10),
20 TIPS (REF FLOW 7).

Characteristics

DEI® Clever Easyflow is a micro-hybrid fluid photo-polymerizing composite. It's a highly aesthetic material, recommended for III and V class restorations, erosions, and I and II classes as integration material of cavities' bases (sandwich technique).

DEI® Clever Easyflow is strongly radiopaque and much filled, therefore it shows special properties of resistance to mastication and erosion. Its indexes of water absorption too are particularly restrained, with undoubted advantage of colour stability. Also available is the transparent incisal type for sealing surface characterizations and transparence effects.

Also available in Blue colour for first layer in aesthetic restoration, first composite layer under temporary splinting, root canal marks, temporary occlusion correction, temporary maintain of intra dental gap.

DEI® Clever Easyflow meets the ISO 4049 regulations.

Composition

Bisphenol A Diglycidyl Methacrylate, Diurethane Dimethacrylate, Triethylene Glycol Dimethacrylate and Ethoxylated Bisphenol A Dimethacrylate, inorganic fillers: Barium Aluminium Silicate Glass, Fumed Silica, Titanium Dioxide and additional substances (photoinitiators, inhibitors, catalysts, stabilizers and pigments) become as a result of free radical polymerization, activated by the ultraviolet light (400nm-500nm).

Technical data

Particle size: 0,9 µm
Total filler (by weight): 62%
Polymerization depth: maximum 2,9 mm
Working time (light exposure): 120"
Photo-polymerization time: 20" for light colours
40" for dark colours
Linear polymerisation shrinkage: 1-1,2%
Compression strength: 315 MPa
Flexural strength: 120,21 MPa
Water absorption: 18,64 µg/mm³
Water solubility: 0,79 µg/mm³

Technical data (BLUE)

Particle size: 0,9 µm
Total filler: 64%
Polymerization depth: maximum 2,5 mm
Working time (light exposure): 120"
Photo-polymerization time: 40"
Linear polymerisation shrinkage: 1-1,3%
Compression strength: 303 MPa
Flexural strength: 105,20 MPa
Water absorption: 19,45 µg/mm³
Water solubility: 0,89 µg/mm³

Technical data (INCISAL)

Particle size: 0,05-1,0 µm
Total filler: 61%
Polymerization depth: maximum 2,9 mm
Working time (light exposure): 90"
Photo-polymerization time: 20"
Linear polymerisation shrinkage: 0,9%
Compression strength: 320 MPa
Flexural strength: 130 MPa

Water absorption: 20 µg/mm³
Water solubility: 5 µg/mm³

Way of use

Prepare the cavity, etch the enamel, rinse and dry well the area, place the adhesive system DEI® Poker Bond or DEI® Extra Bond, following the instructions for use.

For best product performances put on small increases of DEI® Clever Easyflow (max 2 mm), photo-polymerize every layer for 40 seconds. Use the following DEI® Italia protocol for finishing:

1. Put Seal Coat Fast on and photo-polymerize for 60 seconds.
2. Remove Seal Coat Fast from the surface using goat hair and abrasive paste.
3. Polish with cotton pad.

Warnings and precautions

Don't use in cavities previously treated with products containing eugenol, that can inhibit the reaction of polymerization. Don't mix the product with other materials (resins or composites). Don't use on patients who show sensibility to the components. Don't use in wet area. Saliva, blood and water may cause a bad curing on marginal area and inadequate performance of the material. Don't use the product in case of individual sensitiveness. Protect from direct light. Do not use after the expiration date. Bring back to room temperature before use. After use, retract the piston to avoid spillage of product. Avoid contact with skin or eyes. In case of contact, rinse with water and in the presence of symptoms contact a specialist.

Storage

Keep at temperatures not higher than 25°C.

Via Torino, 765 - 21102 Mercallo (VA) - Italy
Phone +39 0331 969270 - Fax +39 0331 969271
www.deitalia.it

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Formas de suministro

2 jeringas de 3,4 g (2 ml) y 6 puntas.

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A2 (REF. FLOW 1), A3 (REF. FLOW 2),
A3.5 (REF. FLOW 9), A4 (REF. FLOW 3),
B2 (REF. FLOW 5), C2 (REF. FLOW 6),
INCISAL (REF. FLOW 8), BLUE (REF. FLOW 10),
20 PUNTAS (REF. FLOW 7).

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