DEI® ETCHING GEL

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DEI® ETCHING GEL

37% ORTHOPHOSPHORIC ACID FOR ETCHING

Product for dental use
Keep away from children
Date of the specifications: 06/2013

Available packages

DEI® Etching Gel (COD. MORD 3):

Four 2 ml syringes + 12 applicator tips

DEI® Etching Gel (COD. MORD 4):

One 60 g syringe + 3 empty 3 ml syringes
+ 50 applicator tips

Characteristics

DEI® Etching Gel is an etching gel, based on a 37% solution of orthophosphoric acid. The extreme thixotropy of the product guarantees maximum efficacy of etching and easiness of application.

Composition

Phosphoric Acid / Water / Silica / Dye.

Indications

Etching of enamel.

Contraindications

Avoid any contact with mucosas, skin, eyes or teeth not requiring treatment.

Way of use

Clean the surface to treat with pumice paste, wash and dry well.

Prepare the cavity planning the chamfering of the enamel edges.

In deep cavities protect the pulp with suitable material.

Etch the enamel applying DEI® Etching Gel, taking care of covering well the chamfered area.

Let take effect for 30-60 seconds.

On more mineralized teeth let the acid work for 60-120 seconds.

Rinse with plenty of spray water and dry off. The enamel should come into sight chalky white.

In case of contamination with water and saliva, it is advisable to repeat the application of gel for some seconds, rinsing and drying well.

Afterwards proceed applying, according the instructions, the products suitable for completion of restoration.

Filling instructions

(DEI® Etching Gel 60 g (COD. MORD 4))

To fill attach the empty syringe, with the plunger down, to 60 g syringe. Gently use the 60 g syringe to fill the empty syringe to desired level.

Separate and reseal as necessary.

Warnings

Avoid contacts of the product with all the surfaces that must not be treated. In case of accidental contact, rinse at once with plenty of water.

In case of contact with eyes, rinse with water and contact a doctor.

Storage and shelf life

Keep at temperatures under 25°C. The applicator can be sterilized in autoclave. Don't use in case of individual sensitivity to components.

Product corrosive for the skin and the surfaces

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