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# **DEI® Rainbow Hydro Fast**

VINILPOLYSILOXANE FOR PRECISION IMPRESSIONS

Product for dental use only.  
Keep away from children.  
Date of specifications: 05/2014

## Available packages

- DEI® Rainbow Hydro Putty Fast:  
two 400 g jars (base + catalyst) (**COD. RA2 1**)
- DEI® Rainbow Hydro Light Fast:  
four 50 ml cartridges (**COD. RA2 2**)
- DEI® Rainbow Hydro Medium Fast:  
four 50 ml cartridges (**COD. RA2 3**)

## Characteristics

DEI® Rainbow Hydro Putty Fast Putty, Hydro Light Fast and Hydro Medium Fast, are highly hydro-compatible vinylpolysiloxanes, with ideal viscosity for all techniques of impression.  
DEI® Rainbow Hydro Putty Fast Putty, Hydro Light Fast and Hydro Medium Fast comply with ISO 4823 regulations.

## Composition

Vinylpolysiloxane materials.

## Way of use

Insert the mixing cannula on the cartridge and extrude directly into the impression a suitable amount of fluid, avoiding incorporation of air bubbles. After use, leave the cannula inserted on the cartridge up to next use for avoiding eventual contaminations of the product, with consequent obstruction of outlet holes.

## Indicative working times

### - DEI® Rainbow Hydro Putty Fast

|                                                       |               |
|-------------------------------------------------------|---------------|
| Mixing:                                               | 30/45 seconds |
| (up to achieve homogeneous coloration of the mixture) |               |
| Working:                                              | 1.30 minutes  |
| Permanence within oral cavity:                        | 2.30 minutes  |
| Hardening:                                            | 4 minutes     |

## Indicative working times

### - DEI® Rainbow Hydro Light Fast

### - DEI® Rainbow Hydro Medium Fast

|                                |              |
|--------------------------------|--------------|
| Mixing:                        | Automix      |
| Total working:                 | 1 minute     |
| Permanence within oral cavity: | 2.00 minutes |
| Hardening:                     | 3 minutes    |

## Warning

The impression can be cast after 2 hours, but it is advisable to wait 12 hours and not go beyond 7 days.

The impression can be electroplated in copper or silver. Working and hardening times refer to use at room temperature. Higher temperatures shorten times, lower ones lengthen them.

## Notes

The use of the products require dispenser gun and cannulae, not included.

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