DEI® CLEVER FIBRE COMPOSITE

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DEI® CLEVER FIBRE COMPOSITE

HIGH RESISTANCE
UNIVERSAL
PHOTO-POLYMERIZING
HYBRID COMPOSITE

Product for dental use.
Keep away from children.
Date of the specifications: 06/2014

Available packages

- **DEI® Clever Fibre Composite LIGHT CURE:** One 4 g syringe in colours: NEUTRAL (COD. COMFC 1), LIGHT (COD. COMFC 2), MEDIUM (COD. COMFC 3), DARK (COD. COMFC 4)
- **DEI® Clever Fibre Composite LIGHT CURE FLOW:** One 4 g syringe UNIVERSAL colour (COD. COMFC 6)

Characteristics and classification

DEI® CLEVER FIBRE COMPOSITE is a product planned and developed for offering exceptional performances of resistance in restorations of areas subjected to strong masticatory stresses. Compared to values shown by other composites, presence of a special filler, **fibres glass-ceramics**, particular silanization techniques used in phase of production, high filler percentage (83% in weight, 71% in volume) give DEI® CLEVER FIBRE COMPOSITE resistance three times higher: 2.8 K1c (Mpa m^{1/2}). Also the retraction to polymerization shows value of great importance (1% in volume!).

Owing to these properties, the technical performances of DEI® CLEVER FIBRE COMPOSITE are not obtainable at present by other marketed composites. DEI® CLEVER FIBRE COMPOSITE is available in four colourings: neutral, light, medium, dark. DEI® CLEVER FIBRE COMPOSITE meets the ISO 4049 regulations.

Indications

- DEI® Clever Fibre Composite LIGHT CURE:

- 1ST and 2ND class restorations
- Restorations of cavities in sandwich technique
- Prosthetic structures for bridges and crowns

- DEI® Clever Fibre Composite LIGHT CURE FLOW:

- 1ST and 2ND class restorations
- Individual fiber post core build up (indirect technique)

Composition

Barium and strontium aluminium silicate glass, borosilicate glass, silica, resins based on Bis-GMA, camphorquinone, pigments.

Particle size: from 0.04 µ to 16 µ.

Way of utilization

Preparation of the cavity

First of all, treat the enamel with water and pumice; isolate with dam the operating area. The preparation must be not contaminated by saliva or other materials. Prepare the cavity seeking for retaining as much enamel as possible. To improve the closing of the edges and the final safety of the restoration, chamfer at 45°-60° the edges.

Preparing rear teeth, avoid to leave enamel without support of dentin.

In deep cavities we recommend the application of CALCIUM HYDROXIDE or DEI® POKER CEM LINE (not included in the package) for protection of the pulp.

Technique of etching

Apply the etching acid only on the ena-

mel, taking care to cover the chamfer and not wet the dentin.

Let the acid take action for 30 seconds. On more mineralized teeth, let the acid "work" for 60/120 seconds. Rinse for 20 seconds. Dry with air.

The etched enamel shall have a chalky aspect; otherwise, or if a contamination by oil, water or saliva has occurred, repeat the procedure.

The etching acid is irritant for skin, mucosae and eyes. Consequently, avoid contacts. Eventually, rinse with plenty of water and, if necessary, call for a doctor.

Technique of adhesive preparation

Put on the dentin (left lightly wet) the dental preparatory DEI® POKER UNIVERSAL PRIME (not included in the package), taking care of its penetration. Dry with air up to complete dryness of the surface.

DEI® POKER UNIVERSAL PRIME is not aggressive for the pulp (does not contain acetone and does not require the etching of dentin for its action); it can be used also in deep cavities (protected as mentioned before) reducing risks of sensitization.

It is very important that before the application of adhesive both dentin and etched enamel must be dry and clean. Put on the adhesive DEI® POKER DC ADHESIVE (not included) evenly on dentin and enamel, spreading it with gentle jet of air not contaminated by oil or water.

CAUTION! The more strong is the air jet, the more reduced is the adhesive ability. Light cure for 20 seconds with halogen lamp.

Application of composite

Apply layers of composite not thicker than 3 mm.

Light cure for 40 seconds.

Repeat the procedure up to full restoration of the tooth.

Finishing technique

Before the photo-polymerization of last layer of composite, apply DEI® SURFACE HARDENER and cure for 40 seconds, wash and dry. The hardening of the composite shall be complete. Using this prod-

uct avoids the necessity of eliminating the superficial layer of inhibition with burs or other products, with shorter work times.

Should increases of appearance and resistance of the reconstruction be wanted, apply DEI® SEAL COAT FAST. Cure for 60 seconds and let polymerize for other 20 seconds. Take away the superficial layer of DEI® SEAL COAT FAST using abrasive paste and small brush made with goat hair, small brush of cotton and DEI® POLISH PASTE.

After finishing, an extremely hard, resistant and shiny surface is obtained. Further, this technique considerably decreases the build-up of tartar and bacterial plaque on the restored tooth.

Cautions

In restorations with composite don't use products containing eugenol. Keep refrigerated at constant temperature of 4°C, or in environment with fresh and constant temperature lower than 25°C. Take the pack out from the refrigerator some hours before use. Don't expose to sources of light or heat.

After use seal again the syringe with its suitable cap.

To release the pressure of the material, turn the piston halfway anticlockwise. Avoid contact with the skin, as in single cases a hypersensitivity may occur: in this case, don't use.

For the utilization of the accessory products, read always the instructions for use of the specific product.

Conservation and validity

If correctly kept at temperatures lower than 25°C.

Via Torino 765 - 21020 Mercallo (VA)
Phone +39 0331 969270
Fax +39 0331 969271
www.deiitalia.it

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