DEI® RAINBOW

HYDRO MEDIUM

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HYDRO MEDIUM

MEDIUM VISCOSITY MATERIAL FOR PRECISION IMPRESSIONS

Product for dental use only. Keep away from children. Date of specifications: 10/2010

Available packages

Four 50 ml cartridges (COD. RAINH 3)

Characteristics

A vinylpolysiloxane with very high hydro-compatibility (contact angle with water at 2 sec. 45°) which ensures the perfect reproduction of sub-gingival areas even in the presence of moisture.

It is indicated for taking precision impressions (together with Putty in one impression setting) or in classic double impression technique. DEI® Rainbow Hydro Medium is a material with medium viscosity, thixotropic (it remains stable without seeping but increases its fluidity when compressed).

It complies with ISO 4823 type 3 regulations.

Way of use

Is packaged in a security cartridge. Place the cartridge on the dispenser and lock it. Insert the cannula and block it with a fourth turn. Extrude the product on an impression tray or directly using an intraoral tip. Insert the impression tray in the oral cavity within 2 minutes from the dispensation and keep it there for 3 minutes. The time reported is indicative: temperatures below 23°C lengthen it, higher temperatures shorten. After use leave the cannula on the cartridge till next use.

Warning

It is suggested to discard the first 2-3 mm of extruded product that could be not fully blended. The impression can be cast after 30 minutes after the removal from the oral cavity. It can be electroplated with copper or silver. It can be sanitized in suitable solutions.

Technical data

Shore A hardness: ab. 55

Storage

Store at room temperature not exceeding 25°C

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