DEI® RAINBOW PUTTY REGULAR

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DEI® RAINBOW PUTTY REGULAR

**HIGH VISCOSITY
VINYL POLYSILOXAN
FOR DENTAL IMPRESSIONS**

Product for dental use only
Keep away from children
Date of specifications: 06/2010

Characteristics

Addition curing putty based on polyvinylsiloxanes.
It complies with ISO 4823 type O regulations.

Advantage

High size stability, optimum consistency, not sticky and easy mixable, odourless and tasteless.

Suggested use and times

With the single spoons take from the jars equal parts of base paste and catalyst paste, mix for 30 seconds without use of latex gloves.

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Introduce the compound inside the oral cavity within 1 minute and 30 seconds from the mixing and leave for 3 minutes until complete vulcanization.

Technical data

Mixing: 30 seconds
Working time: 1.30 (including mixing)
Reticulation: 4.00/4.30 (including mixing)
Shore A 24h: 75

Warnings

Do not contaminate the base paste with catalyst paste.
Seal well the lids soon after the withdrawal of the products.
Don't mistake the lids.
Don't mistake the use of the spoons (the white one is for the catalyst).
The impression may be cast after 30 minutes from vulcanization.
The impression can be sanitized with specific disinfectant solutions.

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