POKER CEM AUTOMIX DUAL

CEMENTO COMPOSITO DUALE

CE
0546

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DUAL COMPOSITE CEMENT

CE
0546

Product for dental use only
Keep away from children
Date of the specifications: 04/2014

Available packages

Two 8.5 g syringe and 10 cannulas
Available in colours: A3 (COD. CEM 1),
NEUTRAL (COD. CEM 4)
Accessories:
50 cannulas (COD. CORE 3)
100 intraoral tips (COD. CORE 1)

Characteristics

DEI® Poker Cem Automix Dual is a dual composite cement of last generation indicated for every kind of cementations based on adhesive technique: veneers, crowns, bridges, fibre posts. It is highly radiopaque and aesthetic. Its straightforward direct supply makes it use extremely handy in every situation.

Owing to high contents of filler (pyrogenic silica and glass, 61 %, very high

percentage for a cement) DEI® Poker Cem Automix Dual shows high resistance to micro-fractures (320 MPa). DEI® Poker Cem Automix Dual owns top technical data, among which stand out very low water adsorption and resistance to transverse forces at level of modern composites.

Composition

Based on Bis-GMA resins, vitreous barium treated fillers, pyrogenic silica, catalysts, pigments.

Instructions for use

Cementation of crowns, bridges, veneers

In case of ceramic materials (veneers or metal free crowns) it is necessary to treat the ceramic surface to be cemented with DEI® Poker Ceramic Prime, let dry and place the dual adhesive DEI® Extra Bond (not included in the package) on both ceramic part and dentin.

In case of cementation of bridges or crowns structured on metal, these items must be prepared with DEI® Extra Bond (Base and DEI® Catalyst blended 15 seconds and applied without photo-polymerization on both metal parts and dentin).

As to veneers, we recommend 20 seconds of polymerization through the facet for improving the polymerization of the cement, that, anyway, shall occur by auto-polymerization in three/ four minutes as it is dual.

Cementation of fibre posts

- Choose the post or the combination of ideal posts whose insertion does not sacrifice tissue.
- Test the post inside the canal seeking for using the size closest to that created during the preparation.
- Etch the canal with DEI® Etching Gel for 15 seconds and rinse for other 15 seconds.
- Dry and proceed applying DEI® Extra Bond within the canal.
- Photo-polymerize and apply a second layer of DEI® Extra Bond, this time preblended with DEI® Extra Catalyst for 15 seconds; don't photo-polymerize; spread the blend also on the post; apply the cement directly in the canal spreading it.
- Insert the post, photo-polymerize and proceed applying the composite material for creating the stump.

All the applications, etching, rinsing, putting on of DEI® Extra Bond and DEI® Poker Cem Automix Dual should be facilitated by using a paste pusher.

Adhesive cementation of sintered zinc oxide

1. Get the laboratory make a sandblast with aluminium dioxide at 200 µ inside the element or bridge.
2. To remove the grease spread orthophosphoric acid DEI® Etching Gel on both the stumps and inside the element, let act 15 seconds. Rinse and dry.

3. Blend DEI® Extra Bond with DEI® Extra Catalyst for 15 seconds and spread with rotating motion on the surface of the stumps and inside the element; don't photo-polymerize.
4. Wait 2 minutes and apply DEI® Poker Cem Automix Dual.

Technical data

Particle size:	from 0.05 to 1.0 µm
Filler:	61% on weight
Working times:	3-4 minutes
Resistance to compression:	320 MPa
Transverse resistance:	130 MPa
Water adsorption:	< 20 µg/mm³
Barcol hardness:	70

Warnings

When not used for long time, keep in freezer.

Avoid contacts with the skin. In single cases hypersensitiveness to components may occur. In that situation, don't use. As to the use of the products accessory to the cement, always follow their specific instructions.

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